

Sechrist Hyperbaric Technical Bulletin 011

Stretcher Headrest Instructions for Use of Built-in Headrest (Stretchers manufactured in 2012 and earlier)

In order to provide additional information on the use of the Stretcher Headrest, below are instructions for use which are currently available in the most recent revision of the User's Manual Hydraulic Gurneys and Stretchers.

The black knobs at the end of the stretcher lock and unlock the headrest mechanism.

To raise the headrest to one of the 14 different angles (maximum 17 inches (43.2 cm) height/30° angle):

Grasp the handle bar and lift up to engage the locking mechanism at the desired position.

To lower the headrest:

Simultaneously turn the right knob clockwise and the left knob counterclockwise to unlock the mechanism and lower the headrest.

All Sechrist Hydraulic Stretchers will now be labeled at the factory or at the time of Annual Service by the Sechrist Hyperbaric Technicians with a label advising users.

