

CARE AND DISINFECTION OF THE CHAMBER AND GURNEY

1. Disinfect the Chamber according to the type of cases being treated and as directed by the medical staff.
2. Wash the Chamber and all Gurney, Stretcher and Mattress surfaces with an approved disinfectant or mild dish washing soap (see Table 7a).

Table 7A: Approved Chamber Disinfectants

DISINFECTANT	MANUFACTURER CITY, STATE	MANUFACTURER PART NUMBER	RECOMMENDED DILUTION WITH WATER	STRENGTH APPROVED FOR USE	APPROVED AT RECOMMENDED DILUTION	APPROVED AT 100% (FULL) STRENGTH
Tor-HB	Ecolab Professional Products Ecolab Inc. St. Paul, MN	N/A	1:64	100%, 1:64	YES	YES
Stat III TB	Ecolab Professional Products Ecolab Inc. St. Paul, MN	N/A	1:128	100%, 1:128	YES	YES
Beaucoup	Ecolab Professional Products Ecolab Inc. St. Paul, MN	N/A	1:128	100%, 1:128	YES	YES
Ascend	Ecolab Professional Products Ecolab Inc. St. Paul, MN	N/A	1:1256	100%, 1:1256	YES	YES
LoH-se	Calgon, Vestal Division STERIS Corp. Mentor, OH	N/A	1:1256	100%, 1:1256	YES	NO

DISINFECTANT	MANUFACTURER CITY, STATE	MANUFACTURER PART NUMBER	RECOMMENDED DILUTION WITH WATER	STRENGTH APPROVED FOR USE	APPROVED AT RECOMMENDED DILUTION	APPROVED AT 100% (FULL) STRENGTH
Sani-Cloth Bleach Wipe	PDI/Nicepak Orangeburg, NY	U13295	Equivalent to 1:10 dilution	Ready to use only	YES	NO
Dispatch with Bleach	Caltech Industries, Inc. Midland, MI	68832	Not provided	Ready to use only	YES	NO
Quaternary Disinfectant Cleaner	ECOLAB® St Paul, MN	6027204	1:256-1:64	1:256, 1:64	YES	NO
Coverage® Spray HB Plus	Steris Corporation Mentor, OH	1624-77	Not provided	Ready to use only	YES	NO
Sani-Cloth® HB	PDI/Nicepak Orangeburg, NY	Q08472	Not Provided	Ready to use only	YES	NO

CLEANING OF PAINTED AND ANODIZED SURFACES

Use the same methods for cleaning as used for the Acrylic Cylinder.