

CARE AND DISINFECTION OF THE CHAMBER AND GURNEY

- Clean the chamber according to the type of cases being treated and as directed by the medical staff. Refer to the list of acceptable cleaners available from SECHRIST INDUSTRIES.
- Wash Stretcher, Pillows, Positioning Pads and Mattress surfaces with mild detergent. Refer to the Gurney User Manual for cleaning instructions.

CLEANING OF PAINTED AND ANODIZED SURFACES

- Use the same methods for cleaning as used for the Acrylic Cylinder.

ACCEPTABLE BIOCIDES per ASME PVHO-2-2019

- Bleach, up to 15% aqueous sodium hypochlorite
- Aqueous hydrogen peroxide, 3% to 20%
- Aqueous chlorine dioxide, up to 2%

APPROVED CHAMBER ACRYLIC CLEANERS

DISINFECTANT	MANUFACTURER	EPA REGISTRATIONNUMBER	RECOMMENDED DILUTION WITH WATER	STRENGTH APPROVED FOR USE	APPROVED AT RECOMMENDE DILUTION
Virasept	Ecolab Inc.	1677-226	Ready to Use	Ready to use	N/A
Oxycide DailyDisinfectant	Ecolab Inc.	1677-237	1:128-1:64	1:128-1:64	YES
Neutral DisinfectantCleaner	Ecolab Inc.	47371-129	1:256-2:64	1:256-2:64	YES
Stat III TB	Ecolab Inc.	39967-89-2677	1:128	1:128	YES
LpH-se	Steris Corporation	N/A	1:256	1:256	YES
Sani-Cloth BleachWipe	PDI/Nicepak	9480-8	Equivalent to 1:10 dilution	Ready to use	N/A
Clorox HealthcareBleach Germicidal Cleaner	Clorox Professional Products	67619-12	Ready to Use	Ready to use	N/A
Quaternary Disinfectant Cleaner	Ecolab Inc.	6836-78	1:256-1:64	1:256 - 1:64 Ready to use	YES
Quaternary Disinfectant Wipes	Ecolab Inc.	6836-372	Ready to Use	Ready to Use	N/A
Coverage® Spray HBPlus	Steris Corporation	6836-152	Ready to Use	Ready to use	N/A
Sani-Cloth® HB Wipe**	PDI/Nicepak	61178-4-9480	Ready to Use	Ready to use	N/A
Sani-Hypercide Wipe	PDI/Nicepak	9480-16	Ready to Use	Ready to use	N/A
Sani-Cloth® Plus	PDI/Nicepak	9480-6	Ready to Use	Ready to Use	N/A
Virex II 256	Diversey	70627-24	1:256	1:256	YES
Virex Plus	Diversey	6836-349	1:128	1:128	YES
Peroxide Multi SurfaceCleaner and Disinfectant RTU	Ecolab Inc.	1677-251	Ready to Use	Ready to Use	N/A
Vital Oxide	Vital Solutions	82972-1	Ready to Use	Ready to Use	N/A
Mikrozyd Sensitive Wipes Premium	Shulke	88494-4, 88494-2, 88494-2-37549, 74559-3, 67619-25, 70627-77	Ready to Use	Ready to Use	YES
Asept HB**	Ecolab Inc	61178 2 1677	Ready to Use	Ready to Use	N/A

***Note: this product has been discontinued by the manufacturer*