

## APPROVED CHAMBER CLEANING AGENTS

It is the user's responsibility to determine the appropriateness of any products prior to actual use and to be cognizant that reformulation of these products by the manufacturer may occur without notice. Compatibility with acrylic, seals, metallic components, paints, and oxygen should all be considered.

- Clean the chamber according to the type of cases being treated and as directed by the medical staff.
- Wash Stretcher, Pillows, Positioning Pads and Mattress surfaces with mild, preferably non-perfumed dishwashing soap in warm water; recommended 100:1 ratio liquid dishwashing soap to warm water. Refer to the Gurney User Manual for cleaning instructions.

### CLEANING OF PAINTED AND ANODIZED SURFACES

- Use the same methods for cleaning as used for the Acrylic Cylinder.

### ACCEPTABLE BIOCIDES per ASME PVHO-2 most recent edition.

- Bleach, up to 15% aqueous sodium hypochlorite
- Aqueous hydrogen peroxide, 3% to 20%
- Aqueous chlorine dioxide, up to 2%

## APPROVED CHAMBER CLEANING AGENTS

DISINFECTANT	MANUFACTURER	EPA REGISTRATION NUMBER	RECOMMENDED DILUTION WITH WATER	STRENGTH APPROVED FOR USE	APPROVED AT RECOMMEND DILUTION
Virasept	Ecolab Inc.	1677-226	Ready to Use	Ready to Use	N/A
Oxycide Daily Disinfectant	Ecolab Inc.	1677-237	1:128-1:64	1:128-1:64	YES
Neutral Disinfectant Cleaner	Ecolab Inc.	47371-129	1:256-2:64	1:256-2:64	YES
Stat III TB	Ecolab Inc.	39967-89-2677	1:128	1:128	YES
LpH-se	Steris Corporation	N/A	1:256	1:256	YES
Sani-Cloth Bleach Wipe	PDI/Nicepak	9480-8	Equivalent to 1:10 dilution	Ready to Use	N/A
Clorox Healthcare Bleach Germicidal Cleaner	Clorox Professional Products	67619-12	Ready to Use	Ready to Use	N/A
Quaternary Disinfectant Cleaner	Ecolab Inc.	6836-78	1:256-1:64	1:256 - 1:64 Ready to Use	YES
Quaternary Disinfectant Wipes	Ecolab Inc.	6836-372	Ready to Use	Ready to Use	N/A
Coverage® Spray HB PLUS	Steris Corporation	6836-152	Ready to Use	Ready to Use	N/A
Sani-Cloth® HB Wipe***	PDI/Nicepak	61178-4-9480	Ready to Use	Ready to Use	N/A
Sani-Hypercide Wipe	PDI/Nicepak	9480-16	Ready to Use	Ready to Use	N/A
Sani-Cloth® Plus	PDI/Nicepak	9480-6	Ready to Use	Ready to Use	N/A
Virex II 256	Diversey	70627-24	1:128	1:128	YES

<b>DISINFECTANT</b>	<b>MANUFACTURER</b>	<b>EPA REGISTRATION NUMBER</b>	<b>RECOMMENDED DILUTION WITH WATER</b>	<b>STRENGTH APPROVED FOR USE</b>	<b>APPROVED AT RECOMMEND DILUTION</b>
Virex Plus	Diversey	6836-349	1:128	1:128	YES
Peroxide Multi Surface Cleaner and Disinfectant RTU	Ecolab Inc.	1677-251	Ready to Use	Ready to Use	N/A
Vital Oxide	Vital Solutions	82972-1	Ready to Use	Ready to Use	N/A
Mikrozid Sensitive Wipes Premium **	Shulke	88494-4, 88494-2, 88494-2-37549, 74559-3, 67619-25, 70627-77	Ready to Use	Ready to Use	YES
Asept HB***	Ecolab Inc	61178 2 1677	Ready to Use	Ready to Use	N/A
Rapid Multi Surface Disinfectant Cleaner RTU	Ecolab Inc.	1677-273	Ready to Use	Ready to Use	N/A
Daily Disinfectant Cleaner Concentrate	Ecolab Inc.	6836-361-1677	1:64 to 1:21.3	1:64 to 1:21.3	YES

\*\* Note: CE Marked product

\*\*\*Note: this product has been discontinued by the manufacturer